

Provox FlexiDerm

How to use Provox[®] FlexiDerm[™]

- For speaking with finger occlusion
- Stick well to the skin
- For a flat stoma


Paulo
User | Proxo




Atos
Coloplast Group

Provox FlexiDerm


How to apply Provox® FlexiDerm™

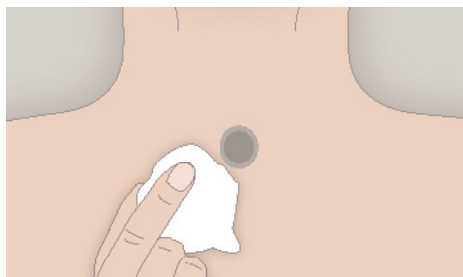


- 1** Clean the skin around your stoma with a Provox Cleaning Towel, or with soap and water. Let it dry.

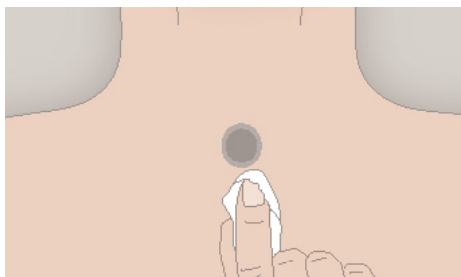


- 2** If your skin is sensitive, use Provox Skin Barrier and let it dry. If you need a stronger attachment, use Provox Silicone Glue or SkinTac. Let it dry for 5 minutes.

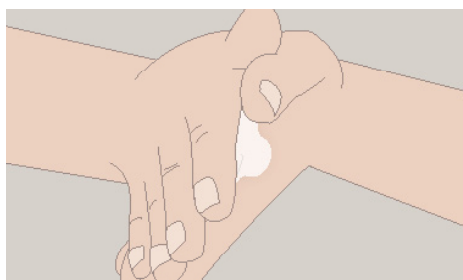


- 3** Rub the adhesive between your hands for one minute to warm it up.

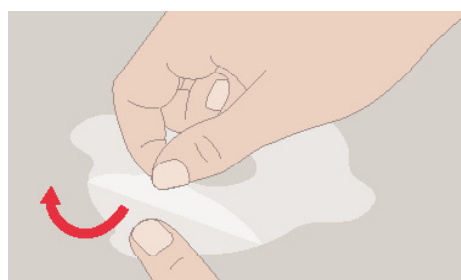


- 4** Remove the protective paper.

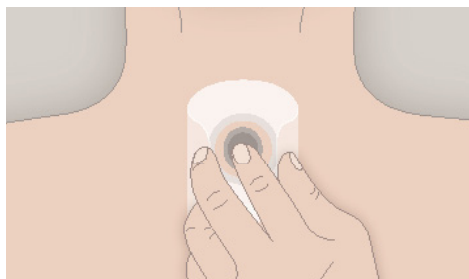

- 5** Fold the adhesive, as shown in picture.

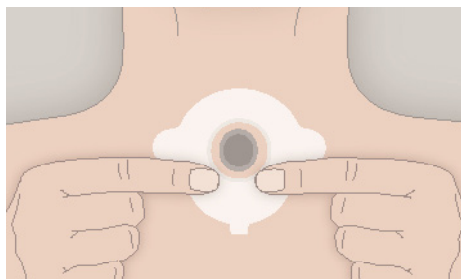

- 6** Attach the bottom section first, followed by the top section, and then the two side sections. Start near the middle of the adhesive, as close to the stoma as possible.

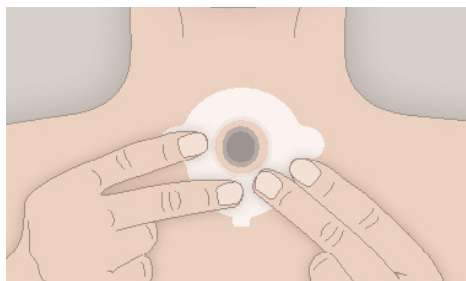


- 7** Massage the adhesive onto your skin.


How to remove Provox® FlexiDerm™



- 8** Apply the Provox Adhesive Remover on top of the adhesive.

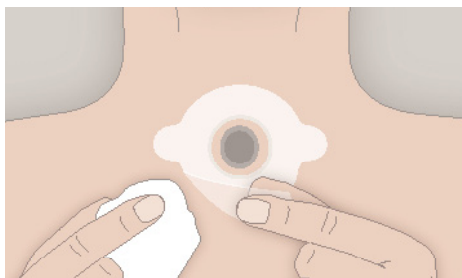

- 9** Lift the finger tab carefully to start taking off the adhesive. Add more Provox Adhesive Remover underneath the adhesive. Then use the wipe to slowly and carefully push the adhesive off your skin.

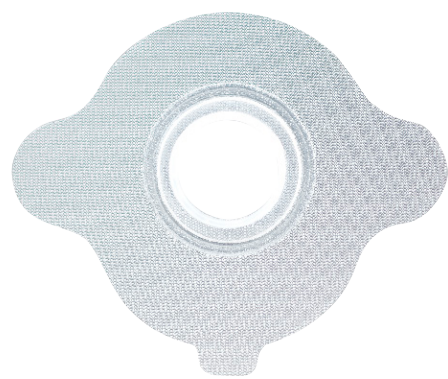

Name	Qty.	Ref. no
Provox FlexiDerm Round	20 pcs	7253
Provox FlexiDerm Oval	20 pcs	7254
Provox FlexiDerm Plus	20 pcs	7331

Contact us for more information

If you want to learn more about Provox FlexiDerm don't hesitate to reach out.

By phone:

By email:

Online:

833.514.2867

info.ca@atosmedical.com

www.atosmedical.ca

Address:

2020 Bristol Circle

Suite #117

Oakville, ON

L6H 0H2