



*Provox*Luna

Peter
User | Proxo

How to use Provox[®] Luna[™]


Better nights for better days

- Sleep comfortably
- Soothe your skin
- Improve your lung health




Atos
Coloplast Group

How to apply the Provox® Luna™ and HME

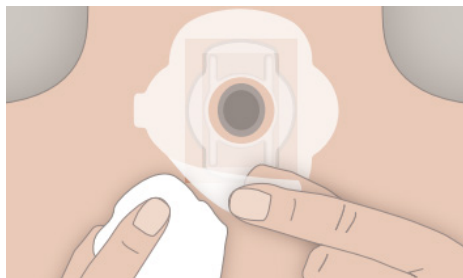


- 1** Gently remove your daytime adhesive. Use Provox Adhesive Remover for easy removal.

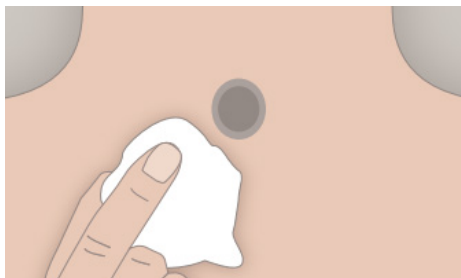


- 2** Clean the skin around the stoma with soap and water. Let it dry. Do not use any skin barrier or cream in combination with Luna.




- 3** Now take the Luna Adhesive out of the packaging. Hold the outer liners and peel away the middle liner.

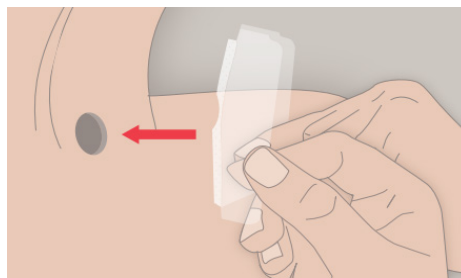

- 4** Apply the ring of the adhesive as closely as possible to the stoma.

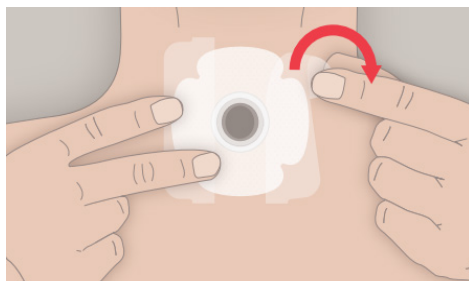


- 5** Massage the middle part onto the skin and peel off the sides. Then massage the entire adhesive onto the skin.

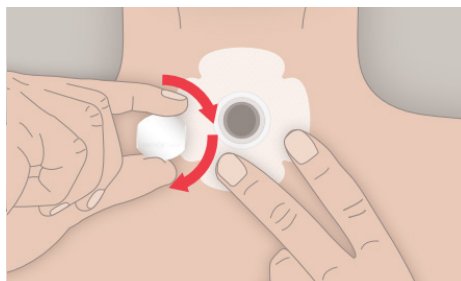


- 6** Put the Luna HME into the adhesive ring with a twisting motion.

How to remove Provox® Luna™

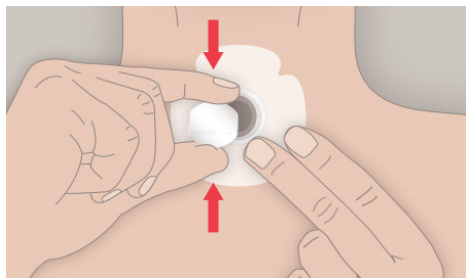

- 7** Gently squeeze the Luna HME to remove it from the Luna Adhesive.

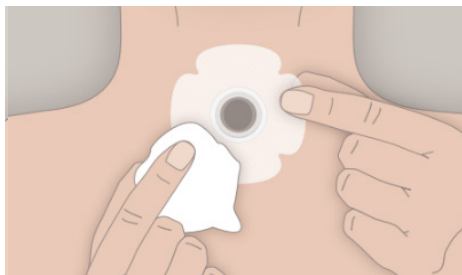


- 8** Carefully remove the Luna Adhesive from the skin. If needed, moisten the Luna Adhesive with water, to make it come loose.

How to speak with the Provox® Luna™ HME



- 9** To speak, occlude the Luna HME by covering the two openings on the sides. Please note, the Luna HME is not designed for speech during daytime.

How to take a shower with the Provox® Luna™ adhesive

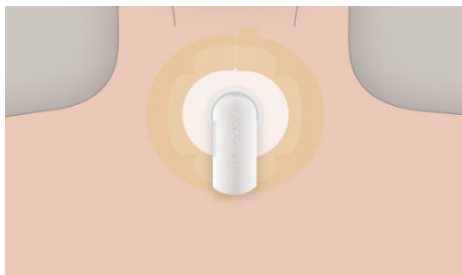

- 10** Replace the Luna HME with the Luna Shower Aid before showering. Use two Provox Adhesive Strips to cover the Luna Adhesive edge, to prevent it from loosening from the skin.



Name	Qty.	Ref. no
Provox Luna HME	30 pcs	8013
Provox Luna Adhesive	30 pcs	8014

Contact us for more information

If you want to learn more about Provox Luna don't hesitate to reach out.

By phone: 833.514.2867
By email: info.ca@atosmedical.com
Online: www.atosmedical.ca

Address: 2020 Bristol Circle
Suite #117
Oakville, ON
L6H 0H2