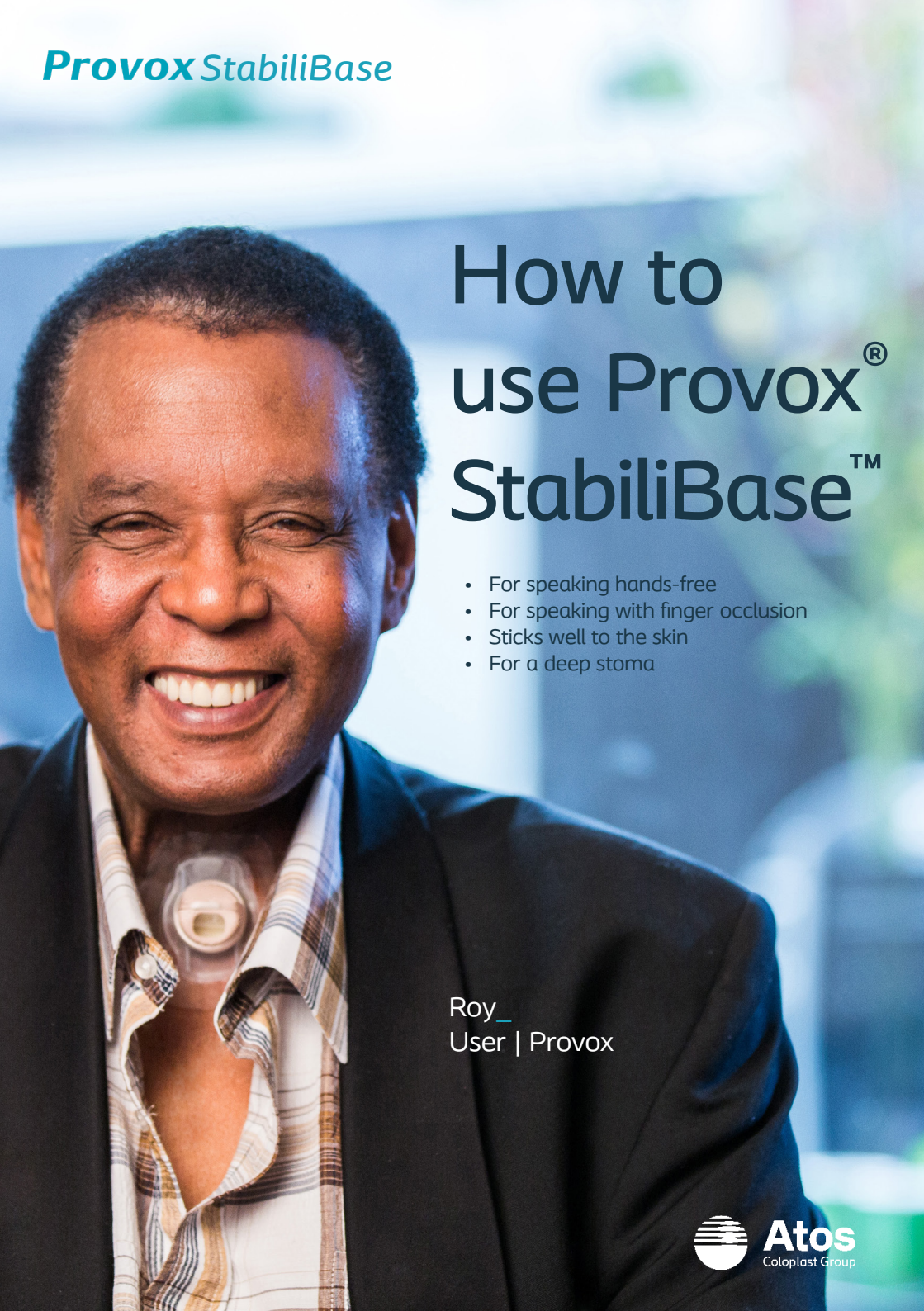


A close-up portrait of a smiling man with dark skin and short hair. He is wearing a dark suit jacket over a light-colored, patterned button-down shirt. A clear, circular Provox StabiliBase speaking valve is attached to his stoma. The background is a blurred indoor setting with blue and green tones.

Provox StabiliBase

How to use Provox[®] StabiliBase[™]

- For speaking hands-free
- For speaking with finger occlusion
- Sticks well to the skin
- For a deep stoma


Roy
User | Provox




Atos
Coloplast Group

Provox StabiliBase

How to apply Provox® StabiliBase™

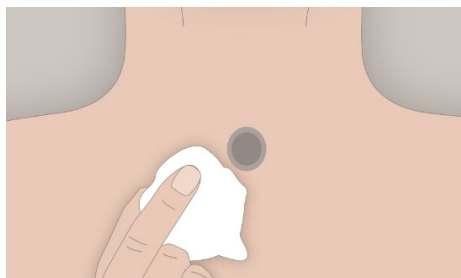

- 1** Clean the skin around your stoma with a Provox Cleaning Towel, or with soap and water. Let it dry.

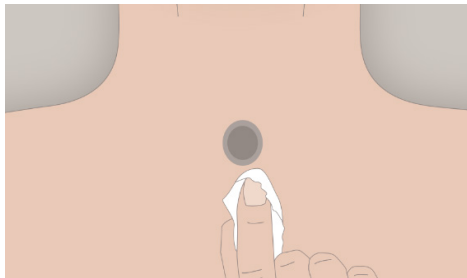


- 2** If your skin is sensitive, use Provox Skin Barrier and let it dry. If you need a stronger attachment, use Provox Silicone Glue or SkinTac. Let it dry for 5 minutes.




- 3** Rub the adhesive between your hands for one minute to warm it up.




- 4** Put the adhesive upside-down on a flat surface. Hold the outer liners down with two fingers, as shown in picture 4. Peel away the middle liner.

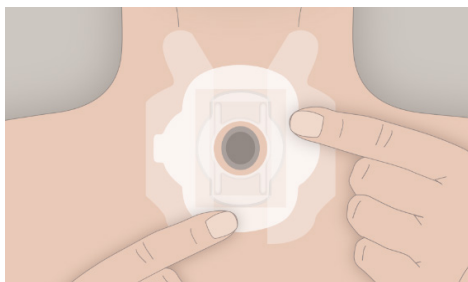


- 5** Apply the middle part as closely as possible to your stoma. If needed, turn it clockwise to fit the shape of your stoma. Massage the adhesive onto your skin.

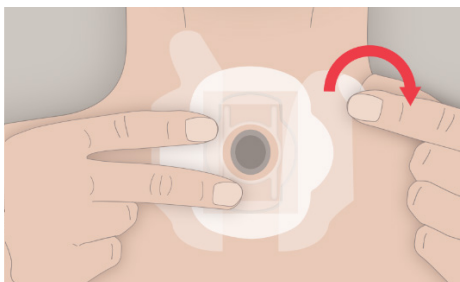

- 6** Peel off one of the outer liners and massage onto the skin. Do the same on the other side.

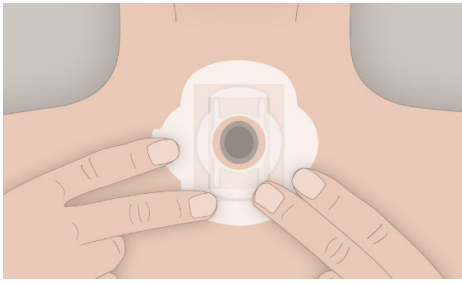


7 Massage the whole adhesive firmly onto your skin.

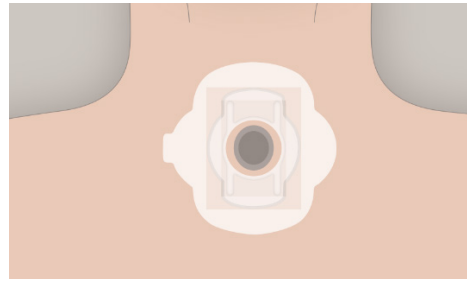


8 If you speak hands-free, don't speak for the first 60 minutes – this makes sure the adhesive sticks properly.

How to remove Provox® Stabilibase™

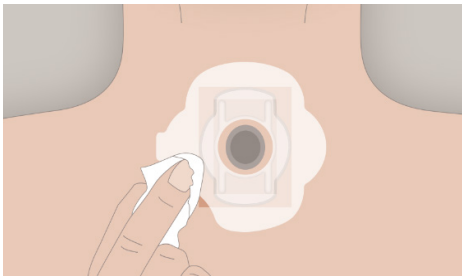

9 Apply the Provox Adhesive Remover on top of the adhesive.

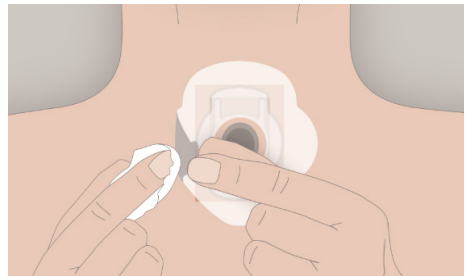

10 Lift the finger tab carefully to start taking off the adhesive. Add more Provox Adhesive Remover underneath the adhesive. Then use the wipe to slowly and carefully push the adhesive off your skin.

Provox StabiliBase



Name	Qty.	Ref. no
Provox StabiliBase	15 pcs	7298
Provox StabiliBase OptiDerm	15 pcs	7318

Contact us for more information

If you want to learn more about Provox StabiliBase don't hesitate to reach out:

By phone:

By email:

Online:

833.514.2867

info.ca@atosmedical.com

www.atosmedical.ca

Address:

2020 Bristol Circle

Suite #117

Oakville, ON

L6H 0H2